
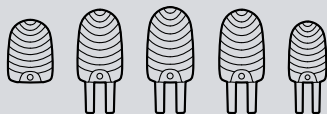


# Quick Start Guide for Clinicians


TASKA HandGen1, Gen2  
Medium (7¾") & Large (8¼") V2.1

 **TASKA**™


# 1. Check

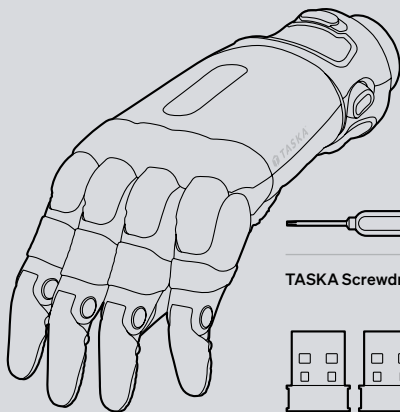


1 x Thumb & 4 x Finger Grip Tips

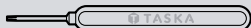

TASKA Screwdriver

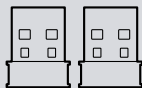


2 x Bluetooth Adaptors

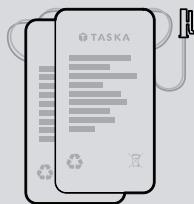


TASKA Battery



2 x Top Faceplates

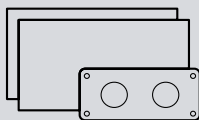

Lamination Dummy Blocks

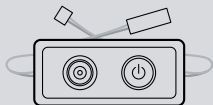


Power Switch




do more


\* Image shows Quick Disconnect Wrist fitting.

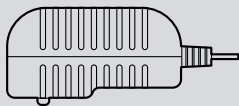

Mains Charger

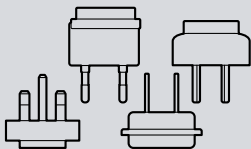


Universal Plug Kit

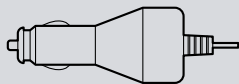


Car Charger

---


## Low Profile Wrist only

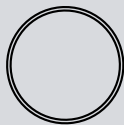


1 x O-ring

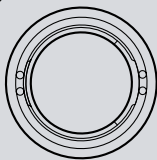

Lamination Ring

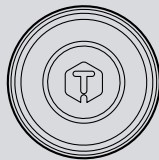

Masking Plug

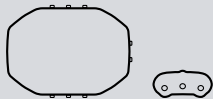

Switch Blocks



Wrist Lubricant



2 x Wrist Set Screws

 **TASKA™**

do more

Items shown are not to scale.

## 2. Download

### Software downloads


All software download links for clinicians and users can be found at:

[taskaprosthetics.com/support](https://taskaprosthetics.com/support)

For more information please contact your local TASKA Representative, or email us at:

[support@taskaprosthetics.com](mailto:support@taskaprosthetics.com)

A free app is also available for mobile.

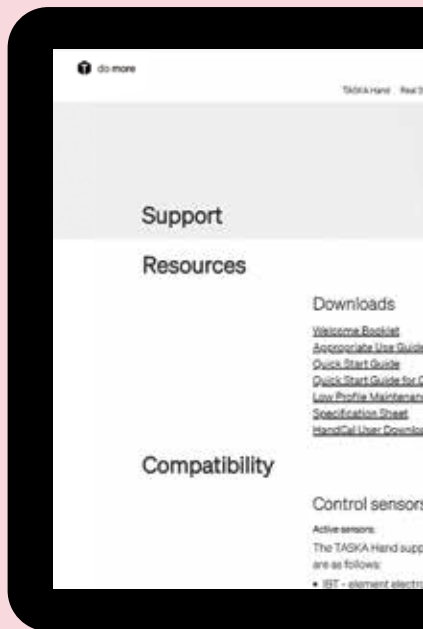

# 3. Control

## Grip Cycle Buttons

For more information on the Grip Cycle Buttons visit [taskaprosthetics.com](http://taskaprosthetics.com)

### Battery light

Light flashes when the battery is low.

### Fault light

Light flashes if there is a fault. Restart the hand.

### Home button

Select the 3 core grips. Press to change between flexi-tool and general grasp.

Hold for pincer grip.



### EMG indicator light

Flashes when inputs are being limited (Grip Triggers or All Sensors).

### Bluetooth light

Illuminated when enabled.

### Hand button light

Flashes briefly when limiting mode is turned on.

### Hand button

Cycle through up to 5 grips of your choice.

Hold to turn Bluetooth on/off.

### EMG button

Change this button's function using the Handcal software to limit Grip Triggers or All Sensor Inputs.

Hold to turn sound on/off.

## 4. Connect (TASKA HandGen1)

---

### Connecting the hand


If the TASKA Hand has been supplied with a Quick Disconnect Wrist, it is easy to attach and remove the hand.

#### To attach the hand:

1. Align the wrist connection of the hand with the socket.
2. Push them together. You will hear a click when the hand has locked to the socket.

#### To remove the hand:

1. Rotate the hand and socket in opposite directions and pull the hand gently away from the socket.
2. When the rotation has reached the correct point, the hand will release from the socket.

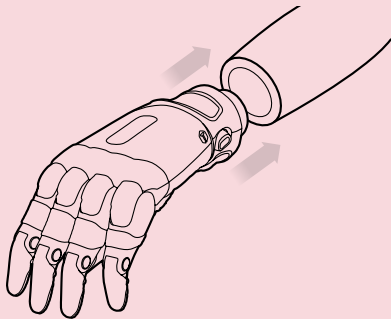

Connecting the Quick Disconnect Wrist.

---

### Low Profile Wrist

For all TASKA Hands that have been supplied with a Low Profile Wrist, please refer to the Low Profile Maintenance & Care Guide for instructions for use.


## 4. Connect (TASKA HandGen2)

### Connecting the hand


If the TASKA Hand has been supplied with a Quick Disconnect Wrist, it is easy to attach and remove the hand.

#### To attach the hand:

1. Insert the hand into the connection on the socket.
2. Rotate the hand towards you 45° (pronate) until it clicks into place.
3. Test the connection by rotating the hand in the other direction.  
The wrist should rotate normally (click) and feel secure.
4. The hand is ready for use.

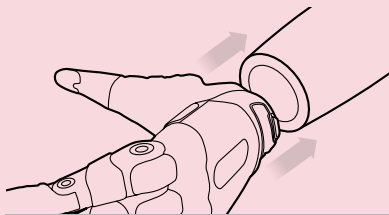

Insert the hand into the wrist connection.

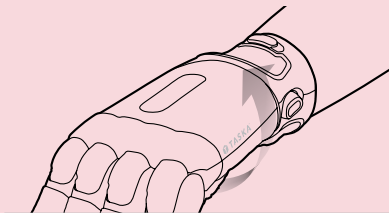

Rotating and attaching the TASKA Hand.




If the hand is fitted with a Seal Ring, clean this and the sealing surface on the socket's wrist connection with a soft damp cloth to remove any dirt before connecting. Refer to the Appropriate Use Guide for more on caring for the wrist.

---

## Disconnecting the hand

### To remove the hand:

1. Press and hold both Release Buttons.
2. While holding down both buttons, rotate the hand away from you 45° (supinate) until you feel resistance. This turn will be smooth and without audible clicks.
3. Remove the hand.

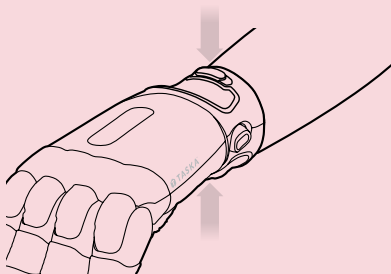


Press and hold both release buttons.

---

## Low Profile Wrist

For all TASKA Hands that have been supplied with a Low Profile Wrist, please refer to the Low Profile Maintenance & Care Guide for instructions for use.

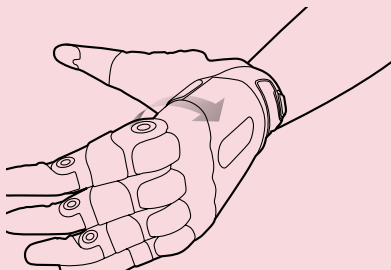

Rotating and removing the TASKA Hand.



## 5. Power


---

### Turning the hand on and off

There is a button on the socket that turns the TASKA Hand on and off.

Before turning on the hand, make sure there is nothing to stop the fingers and thumb from opening fully; this will help to correctly set the position sensors while the hand turns on.

To turn on the hand, press the button once. As the hand turns on, two of the lights on the back of the hand will flash briefly and the fingers will open when the hand is ready.

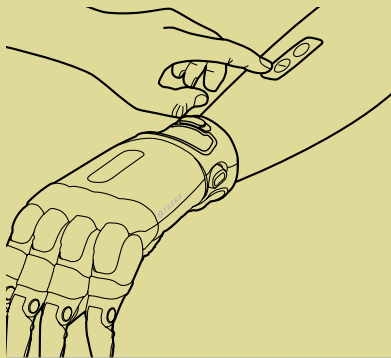

Turning the TASKA Hand on and off.



If using the TASKA Hand with an elbow or another power system, please refer to the relevant user manual for operation.


# 6. Charge

## Battery charging


Remove the hand and arm socket from your residual limb before charging the hand. The charging point is a magnetic socket next to the power button.


### To charge the hand:


1. Place the magnetic end of the charge cord on top of the charging socket. These will automatically align.
2. Connect the charger to either a wall socket for the Mains Charger, or a vehicle power outlet for the Car Charger which does not exceed 12V. During charging, the charger light will change to red. Once complete, the light will change to green.

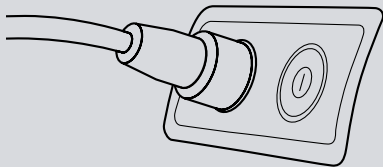

Connecting the charging cable to magnetic charging socket on Power Switch.

 Only use the chargers provided by TASKA to charge the TASKA Batteries.

 **This guide is a quick start use only.** Please refer to the **Appropriate Use Guide** for full detailed guidelines on safe use of the TASKA Hand.

 TASKA Prosthetics, 10 Nelson Street,  
Riccarton, Christchurch 8011, New Zealand.



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
Netherlands